



Vexoderm
Dressing

CLINICAL EVIDENCE VEXODERM DRESSING

Skin and wound care products with
antioxidant and regenerative properties.

www.vexoderm.com

CLINICAL EVIDENCE

LACERATED WOUND DUE TO TRAFFIC ACCIDENT

Vexoderm
Dressing
Gel+

1st Vexoderm

Clinical Cases Contest

Gilda

Cat. 6 years old.

Previous treatment

No previous pathologies.

Dott. Alessandro Negro, DVM

Vexoderm Line Treatment

Vexoderm Dressing is kept in place for 7 days.

Treatment continues with the application of Vexoderm Gel+ every 8 hours by the owner.

BEGINNING



Chronic wound contaminated by rubbing.
Surgical debridement of the lesion is performed.



WEEK 0



Application of Vexoderm Dressing.
Kept in place for 7 days.

WEEK 1



Application of Vexoderm Gel+ every 8 hours by the owner.

WEEK 2



Contraction of the hock wound, which appears drier.

WEEK 3



Complete healing.
Application of Vexoderm Gel+ every 8 hours by the owner continues to improve scar healing.



artinet



vexoderm

CLINICAL EVIDENCE

VERY DEEP EROSIVE LESIONS OF TRAUMATIC ORIGIN

Vexoderm
Dressing
Gel+

1st Vexoderm

Clinical Cases Contest

Tarta

Turtle. 10 years old.

Previous treatment

No previous pathologies.

**Dr.ssa Stefania Di Paco, DVM, Dr.ssa Raffaella Gambirasio, DVM
& Dr.ssa Marzia Possenti, DVM**

Vexoderm Line Treatment

Treatment for 6 weeks with Vexoderm Dressing and Vexoderm Gel+.

BEGINNING



Both forelimbs had very deep and contaminated erosive lesions, with significant soft tissue loss and consequent exposure of forearm bone structures.

WEEK 2



For the forelimbs, application of Vexoderm Dressing every 7 days, covered with greasy gauze, cotton wool, and a self-adhesive bandage.

For the tail, it was decided to use Vexoderm Gel+.



WEEK 6



The gradual and steady reconstruction of the different tissue layers achieved excellent wound healing. Complete healing occurred with the application of Vexoderm Gel+ on the forelimb wounds as well.



artinvet



vexoderm

CLINICAL EVIDENCE

LACERATED-BRUISED WOUND FROM RUBBING

Colony cat

Female. Adult.

Previous treatment:

No previous pathology

Dott. Alessandro Negro, DVM, MSc

Vexoderm Line treatment

Treatment takes 4 weeks, combining Vexoderm Dressing and Vexoderm Gel+. The use of the Vexoderm Gel+ starts at week 3.

BEGINNING



Application of Vexoderm Dressing.

WEEK 1



Cleaning of the wound bed and replacement of the dressing. Very lively granulation. Margins begin to contract.

WEEK 2



Very evident wound contraction. The volunteer proposes to apply Vexoderm Gel+ every 8 hours to avoid further sedation.

WEEK 3



Tissues are closed to healing. Without sedation, getting too close is risky.

WEEK 4



The wound is completely healed.

CLINICAL EVIDENCE

EXPLODED ABSCESS IN THE SACRO-ISCHIAL REGION

Golgo

European Shorthair cat. 9 years old.

Previous treatment:

Attempt to treat it with sugar.

Sedation, trichotomy, washing and cleansing.

Disinfection with chlorhexidine and povidone-iodine.

Surgical debridement and buttonholes for tie-overs.

Dott. Alessandro Negro, DVM, MSc.

Vexoderm Dressing treatment

Application of Vexoderm Dressing and tie-over. Excellent granulation and contraction of margins.

BEGINNING



WEEK 1



WEEK 2



WEEK 2



CLINICAL EVIDENCE

EXPOSED BONE DUE TO TRAUMATISM

Dog, male

Exposed bone due to traumatism.

Previous treatment:

None

Vexoderm Dressing treatment

No. of changes: 4 (once a week).

No. of dressings: 4 Vexoderm Dressing 9x4.

VetCenter Clinic, Bucharest. Dr. Isabelle Caloian.

WEEK 1



WEEK 2



WEEK 4



WEEK 3



CLINICAL EVIDENCE

TRAUMATIC WOUND

Fox Terrier

Dog, male. 10 years old.

Previous treatment:

No previous pathology

Vexoderm Dressing treatment

No. of changes: 3 (once a week).

No. of dressings: 3 Vexoderm 9x4.

Wound closure: week 3.

BEGINNING



WEEK 1



WEEK 2



WEEK 3



CLINICAL EVIDENCE

TRAUMATIC WOUND

Border Collie

Female. 11 months old.

Traffic accident.

Previous treatment

None

Vexoderm Dressing treatment

No. of changes: 3 (once a week).

No. of dressings: 3 Vexoderm 9x4.

Wound closure: week 3.

BEGINNING



WEEK 1



WEEK 2



WEEK 4



WEEK 5



WEEK 6



CLINICAL EVIDENCE

TUMOUR RESECTION

Golden Retriever

Dog, male. 9 years old.

Surgical resection of hepatoid gland tumour in the tail.

Previous treatment

Antibiotics.

Vexoderm Dressing treatment

No. of changes: 2 (once a week).

No. of dressings: 2 Vexoderm 9x4.

Wound closure: week 2.

Prevents potential wound dehiscence.

No maceration

BEGINNING



WEEK 1



WEEK 2



WEEK 11



CLINICAL EVIDENCE

SUPPURATIVE NECROTIC ULCER INFECTED

American Staffordshire

Dog, female. 2 years old.

Previous treatment

Antibiotics / NSAIDs.

Surgical debridement.

Vexoderm Dressing treatment

No. of changes: 7 (first change after 3 days and then once a week).

No. of dressings: 7 Vexoderm 9x4.

Wound closure: week 7.

BEGINNING



WEEK 1



WEEK 2



WEEK 4



WEEK 7



6 MONTHS



CLINICAL EVIDENCE

BURN

Felis Vulgaris

Cat, female. 10 months old. Burn caused by deliberately putting the cat in the oven.

Previous treatment

Moisturising products. Antibiotics.
Surgical debridement.

Vexoderm Dressing treatment

No. of changes: 9 (two changes the first week and then once a week).

No. of dressings: 12 Vexoderm 10x7.
2 Vexoderm 9x4.

Wound closure: week 8.

BEFORE DEBRIDAMENT



BEGINNING



WEEK 1



WEEK 2



WEEK 3



WEEK 4



WEEK 8



WEEK 8



CLINICAL EVIDENCE

TISSUE LOSS DUE TO TRAUMA

European Shorthair

Cat, female. 1 year old.

10 days in evolution

Previous treatment

Antibiotics. Saline cures

Vexoderm Dressing treatment

No. of changes: 5 (once a week).

No. of dressings: 5 Vexoderm 9x4.

Wound closure: week 5.

BEGINNING



WEEK 1



WEEK 1



WEEK 2



WEEK 3



WEEK 4



WEEK 5



WEEK 6



CLINICAL EVIDENCE

POLYTRAUMATISM NECROTIC WOUND

Cat

Male. 7 years old.

Polytraumatism, necrotic wound,
emphysema and edema.

Previous treatment

Suture / Spray (silver)

Vexoderm Dressing treatment

No. of changes: 10 (once a week).

No. of dressings: 10 Vexoderm 9x4.

Wound closure: week 10.

BEGINNING



WEEK 1



WEEK 2



WEEK 3



WEEK 5



WEEK 9



WEEK 10



CLINICAL EVIDENCE

CUT WITH HYPERGRANULATION

Crossbreed horse

Male. 15 years old.

Previous treatment

Antibiotics and cures.

Surgical debridement.

Vexoderm Dressing treatment

No. of changes: 32 (once a week).

No. of dressings: 24 Vexoderm 10x7 and 16 Vexoderm 9x4.

Wound closure: week 32.

Tissue resection at weeks -1, 2, 4 & 5.

BEGINNING



WEEK 1



WEEK 4



WEEK 10




WEEK 17



WEEK 28



6 MONTHS AFTER

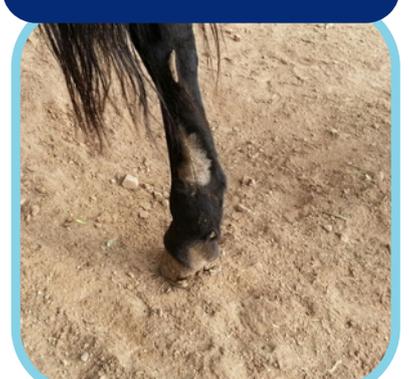

CLINICAL EVIDENCE

CHRONIC WOUND

Thoroughbred horse

Male. 2 years old. 5 months in evolution.

Previous treatment

Antibiotics 2 days before Vexoderm

Vexoderm Dressing treatment

No. of changes: 5 (every 7-10 days).

No. of dressings: 5 Vexoderm 10x7.

Wound closure: week 7.

BEGINNING



WEEK 2



WEEK 3



WEEK 4



WEEK 6



WEEK 7



WEEK 12



CLINICAL EVIDENCE

CHRONIC ULCER

Horse

Chronic ulcer of unknown ethiology.

Previous treatment

Multiple different treatments including honey.

Vexoderm Dressing treatment

No. of changes: 7 (once a week).

No. of dressings: 4 Vexoderm 10x7.

6 Vexoderm 9x4.

Wound closure: treatment with Vexoderm ended before wound closure (week 7).

BEGINNING



WEEK 1



WEEK 2



WEEK 3



WEEK 4



WEEK 6



WEEK 7



CLINICAL EVIDENCE

CUT WITH FLAP

Thoroughbred horse

Female. 22 years old.
5 months in evolution.

Previous treatment

Antibiotics and anti-inflammatory drugs.

Vexoderm Dressing treatment

No. of changes: 7 (once a week).

No. of dressings: 7 Vexoderm 10x7.

Wound closure: week 7.

BEGINNING



FLAP REMOVAL



WEEK 2



WEEK 3



WEEK 5



WEEK 7



CLINICAL EVIDENCE

TISSUE LOSS DUE TO TRAUMATISM

Dog

Female. 3 months old.

Previous treatment

Manuka honey plus antimicrobial cream.

Vexoderm Line treatment



Vexoderm Dressing 10x7

Vexoderm Dressing 9x4 and

Vexoderm Gel+.

Vet4Life Clinic, Bucharest.

BEGINNING

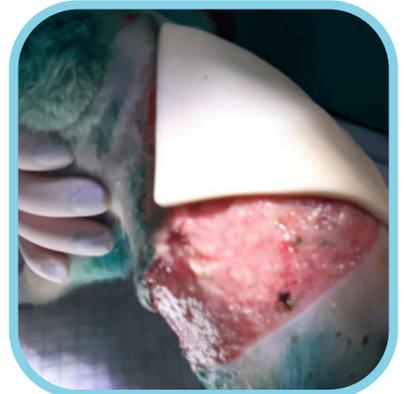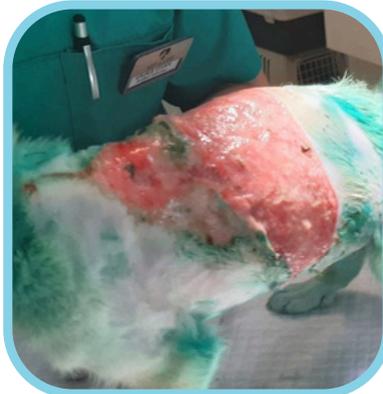

DAY 7



DAY 13



CLINICAL EVIDENCE

ULCER

Bulldog

Dog, female. 4 years old.
Ulcer due to laparotomy.
Possible reaction to suture.

Previous treatment

Systemic antibiotics.
Dressing 6 days after surgery

Vexoderm Dressing treatment

Wound activation and healing.
Treatment with Vexoderm Dressing
until wound closure.

BEGINNING



WEEK 2



WEEK 3



CLINICAL EVIDENCE

CHRONIC SORE

Quarter Horse

Male. 12 years old.

Chronic saddle sore in both sides for over 5 months.

Previous treatment

Ointment

Vexoderm Dressing treatment


Vexoderm Dressing is applied with dressing changes once a week for two weeks until complete closure of the wounds.

A few weeks later hair growth is observed.

BEGINNING



WEEK 1

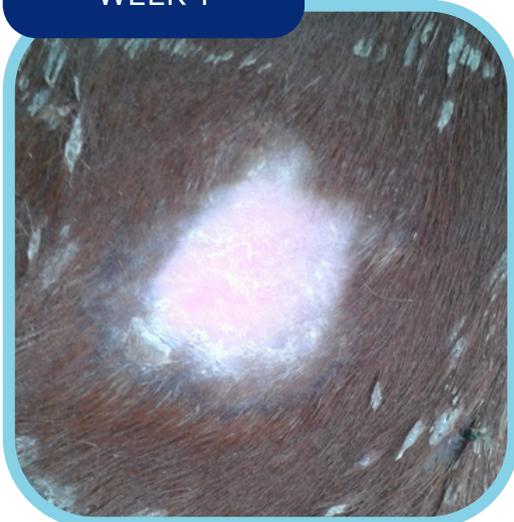

WEEK 11

