

CLINICAL EVIDENCE

VERY DEEP EROSIVE LESIONS OF TRAUMATIC ORIGIN

Vexoderm
Dressing
Gel+

1st Vexoderm

Clinical Cases Contest

Tarta

Turtle. 10 years old.

Previous treatment

No previous pathologies.

**Dr.ssa Stefania Di Paco, DVM, Dr.ssa Raffaella Gambirasio, DVM
& Dr.ssa Marzia Possenti, DVM**

Vexoderm Line Treatment

Treatment for 6 weeks with Vexoderm Dressing and Vexoderm Gel+.

BEGINNING



Both forelimbs had very deep and contaminated erosive lesions, with significant soft tissue loss and consequent exposure of forearm bone structures.

WEEK 2



For the forelimbs, application of Vexoderm Dressing every 7 days, covered with greasy gauze, cotton wool, and a self-adhesive bandage.

For the tail, it was decided to use Vexoderm Gel+.



WEEK 6



The gradual and steady reconstruction of the different tissue layers achieved excellent wound healing. Complete healing occurred with the application of Vexoderm Gel+ on the forelimb wounds as well.



artinvet



vexoderm