

CLINICAL EVIDENCE

LACERATION OF CONNECTIVE TISSUE

Vexoderm
Gel+

1st Vexoderm

Clinical Cases Contest

Corrado

Japanese quail. 1 year old.

Previous treatment

No previous pathology.

Treatment with Vexoderm Gel+

Laceration of connective tissue with exposure of adipose tissue.

The owner applies Vexoderm Gel+ at home twice a day using a cotton swab for 3 weeks.

Dott. Salvatore Ronsisvalle, DMV GPCert(ExAP)

BEGINNING



The patient is sedated, the wound is cleansed, and a proximal suture is applied with resorbable thread.

WEEK 1



The owner applies Vexoderm Gel+ at home, morning and evening, using a cotton swab.

WEEK 2



The owner continues to apply Vexoderm Gel+ twice a day.

WEEK 3



The feathers are starting to grow back. Keeps applying Vexoderm Gel+ until complete healing.

WEEK 4



Healing is completed. The head feathers have completely grown.



artinvet



vexoderm